

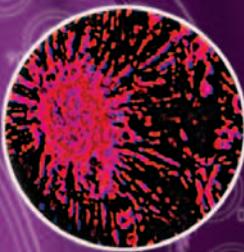


*from the world's first protein
biochip array manufacturer*

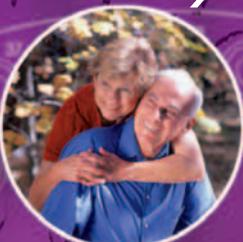

evidence®
biochip array technology

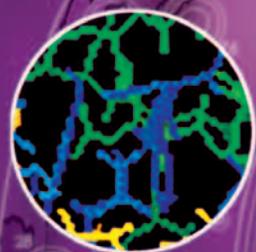

RANDOX

evidence®

biochip array technology

evidence® combines the latest technological advances with innovative system design to present the world's first Protein Biochip Array Technology system. Setting new standards in clinical analysis, the evidence® has the capacity to perform a test output in excess of 1,400 tests per hour. Dedicated software controls all system operations from onboard maintenance to calibration validation, to patient sample profiling for a variety of tests.

The technology is based on a novel concept using a biochip as a reaction platform, with multiple specific ligands (antibodies or antigens) attached at pre-defined sites on the surface. After addition of a patient sample to the biochip, analytes present in the sample bind to the specific ligands. The degree of binding is determined using a chemiluminescent light source and quantified using a Charge Coupled Device (CCD) camera and imaging system.



user features of evidence

**a**

cassette loading tower

Cassettes contain vertically stacked biochip carriers in an easy-load bar-coded cassette. Cassettes are inserted directly and move downward into a storage compartment.

b



refrigerated reagent storage unit

The refrigerated reagent storage unit comprises two separate internal units for panel-specific reagents and chemiluminescent signal reagents. Bar-coded reagents for the test array enable rapid identification.

c

sample unit

Sample tubes or cups can be continuously loaded onto two removable sample carousels (180 capacity) each with a barcode reader for rapid sample identification.

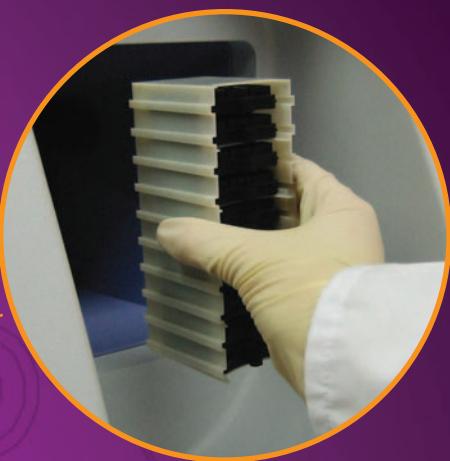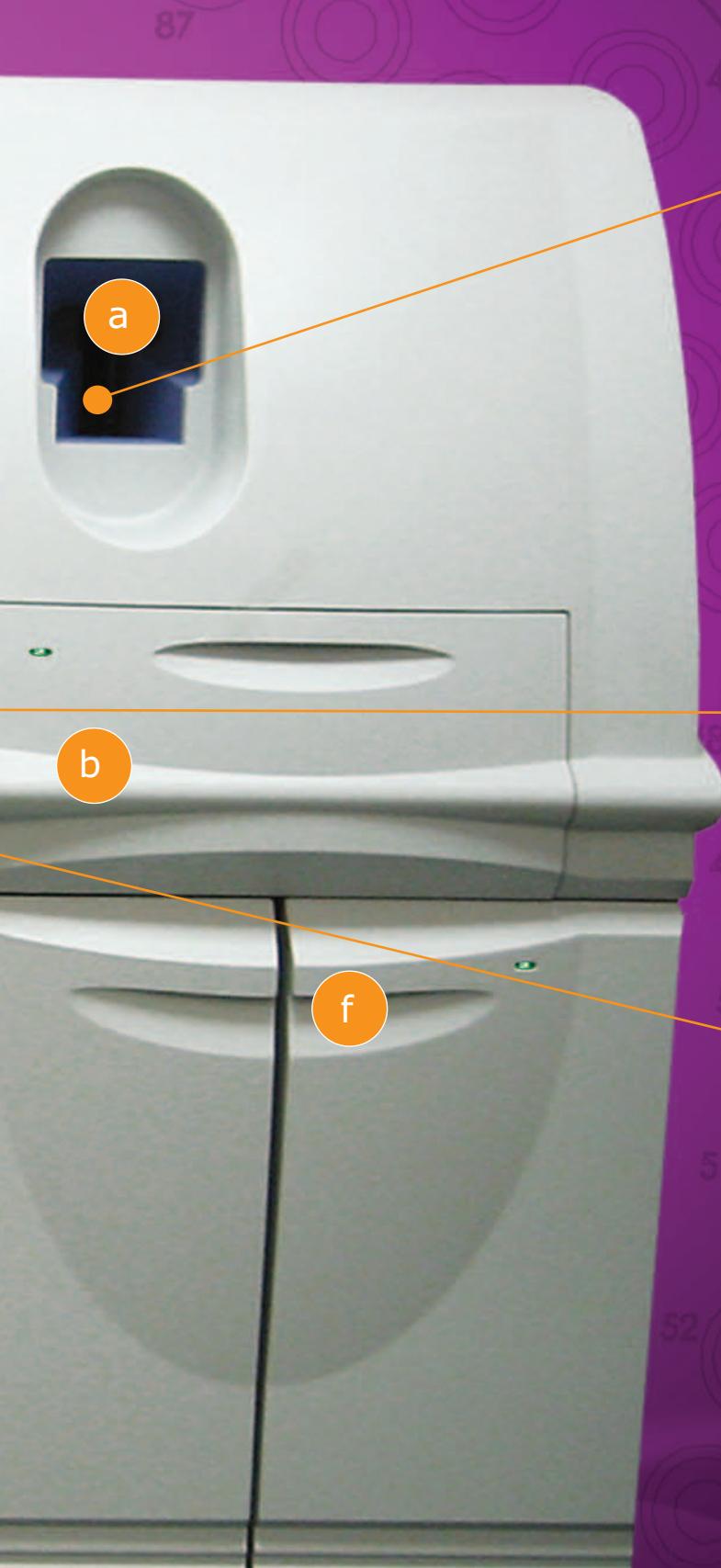

load biochip cassette



load reagents



load samples

d

bulk solutions unit

Universal wash buffer and displacement fluids are required for routine operation of evidence®.

e

cassette collection unit

Cassettes are released into an onboard collection unit when empty or when the test panel changes.

f

biohazard waste unit

An onboard waste container for used biochip carriers minimises the risk of biological contamination and contains an interlocked door for enhanced safety.


evidence® management system

All operational functions of evidence® are managed and controlled by the dedicated Windows® based system software. System sensors enable regular update of onboard inventory details, which are constantly displayed on the user screen.

The Graphic User Interface (GUI) comprises a main access screen, three easiview inventory screens and a menu options section. Easiview screens are used to provide constant on screen information of inventory status. The three screens provide information on the Active Sample Ring, Panel-specific reagents and Waste and Signal reagents.

Software features

- Colour-coded, on screen inventory units for demonstration of component status.
- Automatic system initialisation procedure.
- Retrospective reporting facility to enable the retrieval of previously unreported tests.
- Calibration information supplied on disc for easy upload.
- User-defined cut-off values for semi-quantitative assays without the need to recalibrate.
- Multi-format option for results review e.g. by patient sample, by biochip carrier, etc. This facilitates troubleshooting of results.
- Results archive facility.
- Access to analyser software for remote maintenance and trouble shooting.
- Test selection and reporting option from dedicated assay panels to eliminate test redundancy.
- Automatic onboard inventory check for lot and panel specific biochips and reagents before a test panel is presented for selection.
- To ensure safe operation and handling, software restricts access to onboard units and reagents.
- Audible alarms to alert user
- evidence® can be connected to a laboratory information management system (LIMS)
- QC software with Multi-rule capability

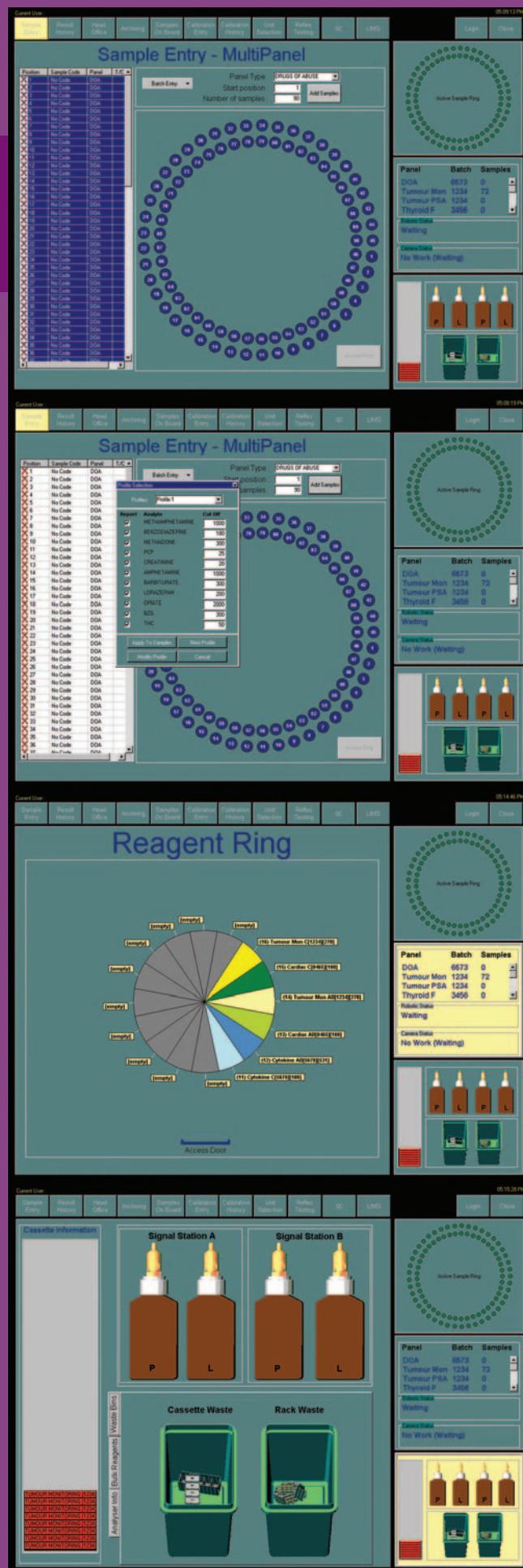


evidence®
biochip array technology

A biochip's journey through
evidence®
is fully automated




ready to use, 9 patient biochip carrier

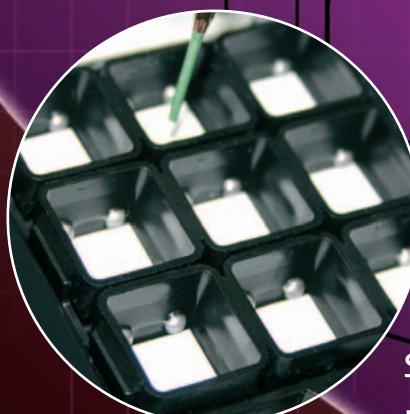

sample & reagent addition



RANDOX

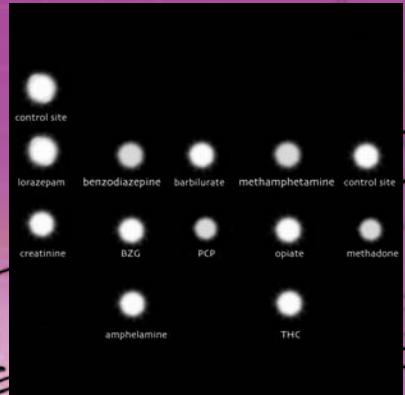

discrete sites on
each biochip quantify
individual analytes



CCD image capture
of 9 biochips in carrier

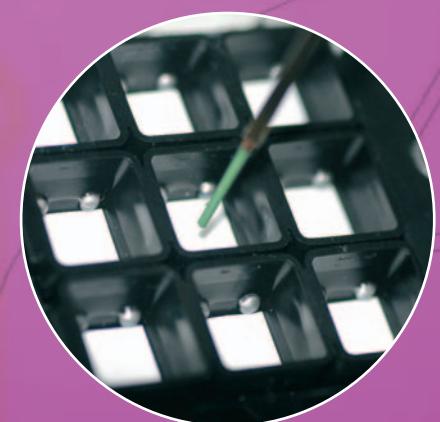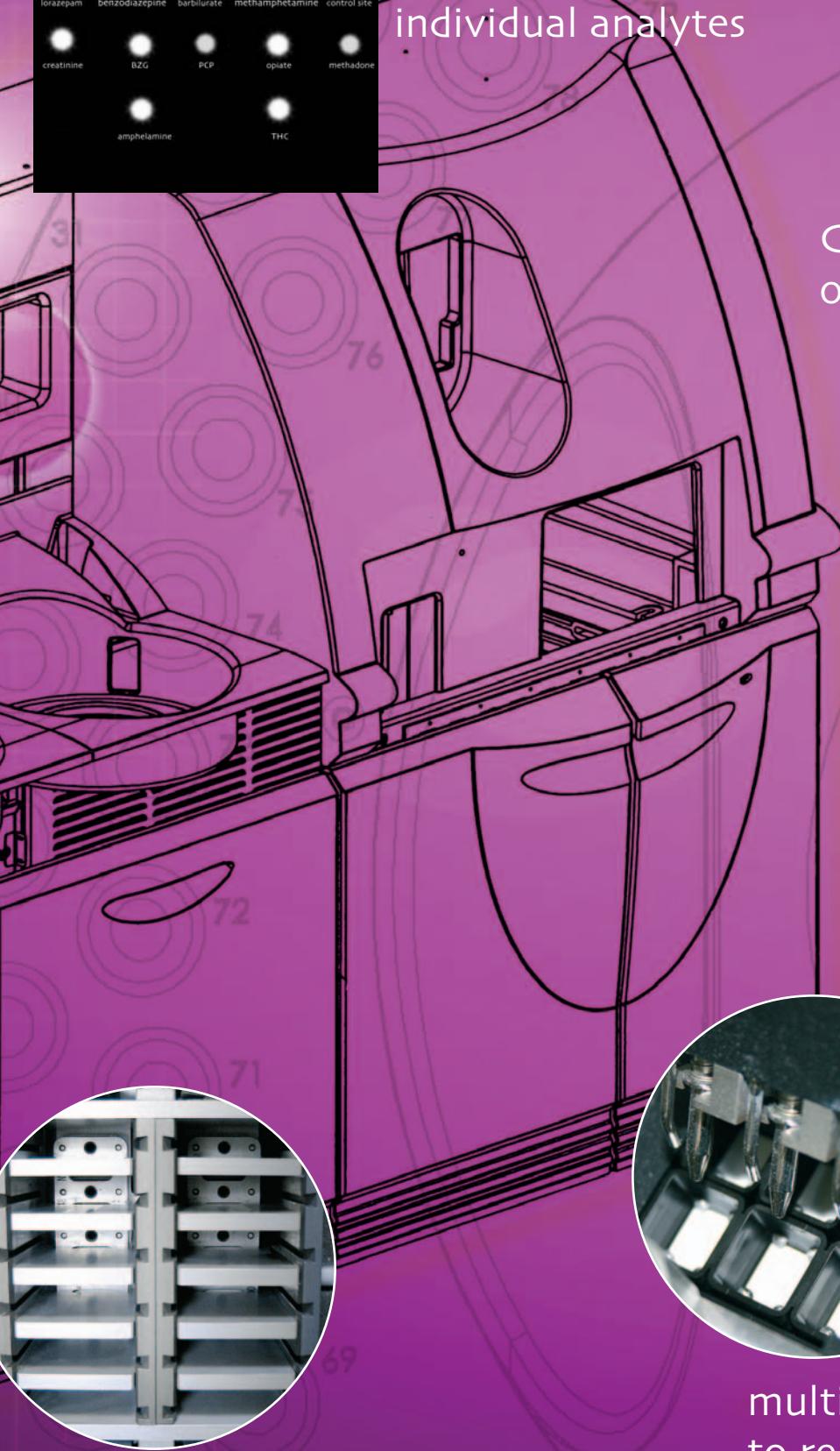

signal reagent
addition



multiple washes
to remove excess
reagent

incubation at 37°C

multiple test biochips comprising up to 25 discrete test regions on each biochip



Consolidation of Tests

evidence® utilises biochips as platforms for immunoassay measurement. Specific and simultaneous profiling of biological markers is the biggest advantage of protein biochip array technology for many applications.

RANDOX

International Headquarters Randox Laboratories Ltd.,
55 Diamond Road, Crumlin, Co. Antrim, United Kingdom, BT29 4QY
t: +44 (0) 28 9442 2413 e: marketing@randox.com
www.randox.com



2001



Clinical Arrays

Free Thyroid Array

- Thyroid stimulating hormone (TSH)
- Free tri-iodothyronine (FT₃)
- Free thyroxine (FT₄)

Total Thyroid Array

- Thyroid stimulating hormone (TSH)
- Total tri-iodothyronine (TT₃)
- Total thyroxine (TT₄)

Fertility Array

- Follicle stimulating hormone (FSH)
- Prolactin (PRL)
- Lutenising Hormone (LH)
- Estradiol
- Progesterone
- Testosterone

Tumour Monitoring Array

- Carcinoembryonic antigen (CEA)
- α- Fetoprotein (AFP)
- Human chorionic gonadotrophin (hCG)
- Gastrointestinal tumour antigen (CA 19-9)
- Ovarian tumour antigen (CA 125)
- Breast cancer antigen (CA 15-3)

Tumour PSA Array

- Total prostate specific antigen (tPSA)
- Free prostate specific antigen (fPSA)
- Carcinoembryonic antigen (CEA)

Adhesion Molecules Array

- I CAM - 1
- V CAM - 1
- E - Selectin
- P - Selectin
- L - Selectin

Cytokines & Growth Factors Array

- Interleukin-1 alpha (IL-1 α)
- Interleukin-1 beta (IL-1 β)
- Interleukin-2 (IL-2)
- Interleukin-4 (IL-4)
- Interleukin-6 (IL-6)
- Interleukin-8 (IL-8)
- Interleukin-10 (IL-10)
- Vascular Endothelial Growth Factor (VEGF)
- Tumour Necrosis Factor-alpha (TNF-α)
- Interferon-gamma (IFN-γ)
- Epidermal Growth Factor (EGF)
- Monocyte Chemotactic Protein-1 (MCP-1)

Cardiac Array

- Troponin I
- Myoglobin
- CK-MB
- Carbonic Anhydrase III (CA III)
- Glycogen Phosphorylase BB (GPBB)
- Fatty Acid Binding Protein (FABP)

Drugs of Abuse Array I

- Amphetamine
- Methamphetamine
- Cocaine
- Barbiturates
- Cannabinoids
- Opiates
- Methadone
- Benzodiazepine I
- Benzodiazepam II
- Phencyclidine
- Creatinine

Multi-analyte Quality Controls

- Drugs of Abuse
- Cardiac
- Cytokine
- Immunoassay
- Customised quality controls
- Adhesion Molecules

Future Developments

- Allergen Array (IgE)
- Cerebral Arrays
- Cytokine Arrays
- Maternal Screen Array
- Breast Cancer Diagnosis and Classification
- Bone Marker
- Drugs of Abuse Array II